

**Trainer for EVZIO®**  
**Trainer Instructions for Use**

**Important:**

**The Trainer for EVZIO Does Not contain a needle or medicine. Always carry your real EVZIO with you in case of an opioid emergency.**

Tell your family, friends, co-workers or other individuals who may need to use EVZIO during an opioid emergency where you keep your EVZIO.

**Important Information about the Trainer for EVZIO:**

Inside your Trainer for EVZIO are:

- batteries
- a speaker that will make a beeping sound and that produces electronic voice instructions
- red and green blinking lights

The Trainer for EVZIO batteries are made to last for over 1,000 demonstrations or practices.

If the electronic voice instructions do not work properly, the Trainer for EVZIO can still be used for demonstration or practice. If this happens, use the instructions below and follow the written instructions on the Trainer for EVZIO label.

**What is the Trainer for EVZIO?**

- The Trainer for EVZIO does not contain a needle or medicine and can be reused to practice your injection.
- Practice with the Trainer for EVZIO before an opioid emergency happens to make sure you are able to safely use the real EVZIO in an emergency.
- A Trainer for EVZIO comes with each EVZIO prescription so that you and your caregiver can practice and demonstrate how to use EVZIO.

Figure A

Trainer for EVZIO



Trainer      Trainer Outer Case

Trainer for EVZIO:

- is inside a **white and black** outer case
- **does not** contain a needle or medicine
- **can be** reused (the **Red** safety guard can be placed back on the **Black** base after use)

EVZIO 2 mg



EVZIO 2 mg      EVZIO 2 mg Outer Case

EVZIO 2 mg:

- is inside a **blue and purple** outer case
- contains a needle and medicine
- cannot be reused (the **Red** safety guard cannot be placed back on the **Black** base after it is removed)
- has an expiration date

**In case of an opioid overdose or possible opioid overdose emergency, use the real EVZIO and not the Trainer for EVZIO.**

### Who should practice using the Trainer for EVZIO?

Anyone who may need to help you with EVZIO in case of an opioid overdose or possible overdose emergency should practice using the Trainer for EVZIO.

Have them practice using the Trainer for EVZIO and review the Patient Information leaflet included in the packaging with your prescription of EVZIO.

For more information and video instructions on the use of EVZIO, go to <http://www.EVZIO.com> or call 1-855-77-EVZIO.

### Practicing with the Trainer for EVZIO

- Practice with the Trainer for EVZIO before an opioid emergency happens to make sure you are able to safely use the real EVZIO in the case of an opioid overdose or possible overdose emergency.
- You and your caregivers should practice every day for the first week after you receive your Trainer for EVZIO, and then at least 1 time each week, to help you feel familiar with using EVZIO quickly and safely during an opioid overdose or possible opioid overdose emergency. Even when you are familiar with using the Trainer for EVZIO, continue to practice using it often.

### How to use the Trainer for EVZIO

- Even though the Trainer for EVZIO does not have a needle and contains no medicine, it works the same way as the real EVZIO.
- Just like the real EVZIO, the Trainer for EVZIO contains an electronic voice instruction system to help guide you through each step of the injection. If the voice instructions do not work for any reason, you can still use the Trainer for EVZIO to practice using the instructions below and following the written instructions on the Trainer for EVZIO.
- The Trainer for EVZIO has the same blinking red and green lights as the real EVZIO. These blinking lights help provide visual cues for each voice instruction and step.

#### Follow these steps to practice using the Trainer for EVZIO

1. **Pull the Trainer for EVZIO from the outer case.** See Figure B.
2. **Pull off Red safety guard.** See Figure C.

Figure B



Figure C



Note: The **Red** safety guard is made to fit tight similar to the safety guard on EVZIO. **Pull firmly to remove.**

3. Place **Black** end of the Trainer for EVZIO against the middle of the outer thigh (through clothing, if needed), then press firmly, and hold in place for 5 seconds. See Figure D.

Figure D



Only practice using the middle of the outer thigh. The outer thigh is where you would inject with the real EVZIO.

**Note:** The Trainer for EVZIO makes a distinct sound (click and hiss) when you press it against the outer thigh. This is the same sound that is made with the real EVZIO. This is normal and indicates EVZIO is working correctly. Do not pull the Trainer for EVZIO away from the leg when you hear the click and hiss sound.

4. After practicing, reset the Trainer for EVZIO:

- Replace the Red safety guard. See Figure E.
- Slide the Trainer for EVZIO all the way back into the white outer case to reset the electronic voice system. See Figure F.

Figure E



Figure F



**Note:** Do not hold the Black base when replacing the Red safety guard. If you do this, the Black base may not reset properly and may prevent you from inserting the Red safety guard into the Black base. If this happens, remove the Red safety guard and repeat Step 4 above.

Leave the Trainer for EVZIO in its outer case for at least 5 seconds between each time you practice to allow the electronic voice system to reset.

**How should I dispose of the Trainer for EVZIO?**

The Trainer for EVZIO contains electronics and lithium coin cell batteries, and should be disposed of in the correct manner. Follow your State and local environmental regulations for disposal.

For California Only: This product uses batteries containing Perchlorate Material- special handling may apply. See [www.dtsc.ca.gov/hazardouswaste/perchlorate](http://www.dtsc.ca.gov/hazardouswaste/perchlorate)

For more information or questions about the Trainer for EVZIO, go to [www.EVZIO.com](http://www.EVZIO.com) or call 1-855-773-8946.

**How should I store the Trainer for EVZIO?**

- Store the Trainer for EVZIO at room temperature between 59°F to 77°F (15°C to 25°C).
- Store the Trainer for EVZIO in its outer case.

This Instructions for Use has been approved by the U.S. Food and Drug Administration.

Manufactured for kaleo, Inc. Richmond, VA 23219

Revised: 10/2016